ITA

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Prodotto

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Indicazioni

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Surgical accessories.

Indications

Preparation of the surgical site for Endosseous implant insertion. The surgical accessories of ITS implant system (surgical drills. Driver, Connectors, Tappers, Patchets, etc.) were specifically studied to be used during the preparation phase of the implant site and for the insertion of ITS implants. For complete details of the components please refer to general ITS product catalogue.

Preservation

Preserve avoiding bumps which could damage the cutting edges. Preserve the surgical drills and tappers always clean and in a dry place.

Precautions for use drills

Use the drill setting the handpiece rotations in order to absolutely avoid overheating the bone (47°C). While using, alternate the drilling procedure cooling by irrigating the implant site with plenty sterile physiological solution. The device for design and sales are destined to be used by specialized medical personnel. ESSEMME COMPONENTS S.r.l. is not responsible for improper use.

General information

Before inserting the implants in the patient's mouth carefully read the instructions for use, contained in the packages of the implants themselves. Following are some indications relative to use of specific tools.

Drills (class IIa)

We recommend the use of a low speed handpiece to avoid developing bone necrosis. In general you must avoid abrupt speed changes. Never apply a pressure that stop the instrument rotation. This could cause excessive heat in the tissues and ruin both the instrument and the handpiece used (contra angle). We recommend to use opportune cooling and if using a drill with internal irrigation, make sure you insert correctly the head guide needle in the hole where the drill passes. Incorrect insertion may cause vibration of the instrument, eccentric rotations and premature wear that may cause the drill to bend. We recommend you use only suitable contra-angles for surgical purposes. Have them periodically checked in accordance with the instructions of the manufacturers to prevent malfunction. For major safety and caution towards wear resistance, we recommend that the drills be used no more than 15 working cycles. ITS will not assume responsibilities in case of excessive use. The drills must never be sharpened before use.

Tappers (class IIa)

Even if ITS implants have a conformation that it is easy to insert and screw, we recommend tapping in case of thick bone. The ITS implant system foresees a different tapper for each implant morphology. Tappers may be used manually with a ratchet or a mechanical driver.

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Driver (class I)

ITS drivers are available in two different lengths, which are compatible with all components destined to be screwed on the implants (screw tap, healing caps, transfer screws, abutment screws). Before screwing, make sure you insert the hexagon wrench from the tip of the drivers in the screw's hexagon, otherwise you risk breaking the drivers to reduce risk tied the hexagon's wear.

Ratchet, depth, gauges, castables etc. (class I)

There are no particular precautions, different from those that are normally indicated in normal dental procedures.

Tray/organizer (class I)

The ITS system offers a practical tray in which you may preserve your surgical tools. The organizer is made up by a container that can be opened, in which there is a tray with specific spaces for each single instrument for standard surgical procedures.

Precautions

Wash and sterilize before use. Surgical drills and tappers may undergo maximum 15 sterilization treatments.

Sterilization

The drills must be accurately cleaned from eventual pieces of residual bone. Wash with a disinfecting detergent in ultrasound container. Sterilize in autoclave at a temperature of 134°C for 20 minutes at the pressure of 2 bar for metallic parts only.

Cleaning procedure

Cleaning: clean in a 0.5% aqueous solution of concentrated detergent based on a multi-enzyme system using lukewarm running water (35 ° C - 40 ° C). Brush with a soft bristle brush until there is no visible evidence of debris or dirt. Leave to soak for 10 minutes.

Rinse: rinse with warm running water (50 ° C - 60 ° C) visually checking the absence of residues, including detergent.

Disinfection

Dry with a sterile cloth soaked in 70% ethyl alcohol-based disinfectant.

All surgical components of the ITS implant system are reserved for professional use with personnel having the necessary qualification.

It is necessary to report any serious accident that occurred in relation to the medical device supplied by us to the manufacturer and to the competent authority of the Member State in which it is located.

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